


Mar Egeo

8230

INSTRUCTION MANUAL

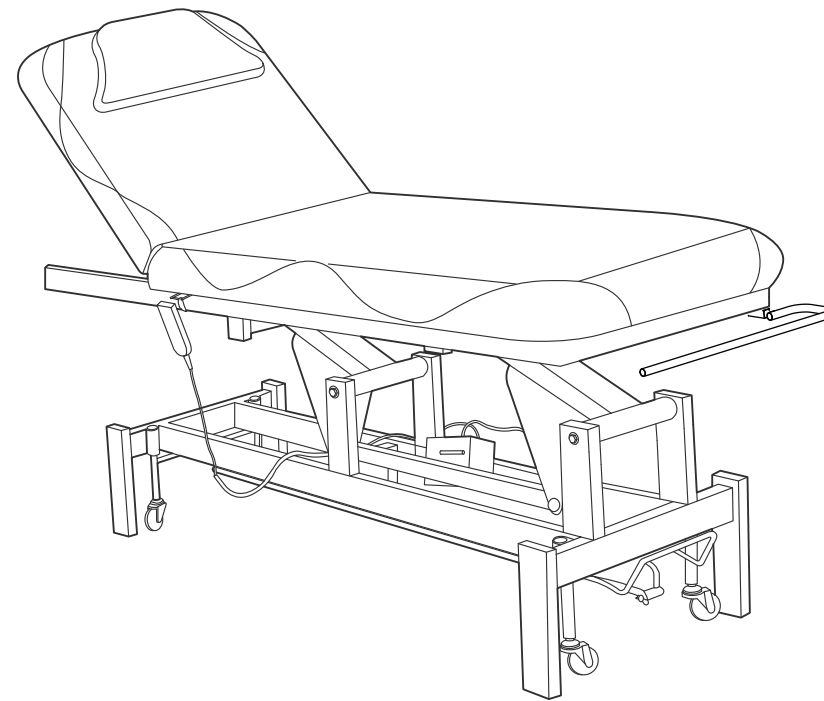

EN

CONTENTS

Product Introduction	3
Warranty	4
Product Information	5
Operation Instructions	9
Control Panel	11
Maintenance	13
Troubleshooting Guide	14
Aftercare Instructions	15
Contact Us	16

Please read the operation manual before using the product.

Please put the operation manual in the place easy for you to read and you can read it at any time.

Product Name: Mar Egeo
Product Code: 8230

Thank you for placing your order with the DIR In Reality Group. Our materials inc. faux leather etc, are specially sourced to provide quality and performance, as well as meeting UK/EU safety regulations to ensure customer satisfaction.

All of our facial beds are produced with strong resilient faux leather, tested by our manufacturer to ensure strength, memory, heat resistance and stain proofing. We also fit our facial beds with fire-retardant, pre-moulded memory foam to ensure continual use and sustained durability which we offer a 1 year warranty to ensure customer confidence in our products and our company.

Mar Egeo facial bed is easy to work with and comfortable to use. The high-density memory foam and remote controlled height of the bed make it a pleasure for you and your clients.


WARRANTY

The warranty period is one year. DIR warrants to you, the original buyer, that goods shall be free of defects in material and/or workmanship attributable to manufacture for one year from date of buyer's receipt of product. Damage resulting from improper handling after merchandise is received, or from exposure to weather or extreme atmospheric conditions is not covered by the warranty.


For any other information please contact our support team:

DIR BIRMINGHAM	DIR LONDON	DIR NEW YORK	DIR COLOGNE	DIR IRELAND	DIR CHINA
66 Rea Street South Birmingham B5 6LB UK ----- Tel: +44(0)121 6434777 ----- Email: sales@dirgroup.co.uk	Unit 1 River Road Business Park 33 River Road, Barking Greater London IG11 0DA UK ----- Tel: +44(0)20 8591 9310 ----- Email: sales.london@dirgroup.co.uk	24 Woodward Ave. Suite C Queens, NY 11385 USA ----- Tel: +1 (718)628-6668 +1 (800)728-2008 ----- Email: sales@dirsalonfurniture.com	Konrad-Adenauer-Straße 25 50996 Köln Germany ----- Tel: + 49 (0) 221 4678 0417 ----- Email: sales@dirgroup.de	Unit 5 Block E, Riverview Business Park, Nangor Road, Dublin 12 Republic of Ireland ----- Tel: +353 (01) 429 7969 ----- Email: sales.ireland@dirgroup.ie	广东伊众健康科技有限公司 Guangdong Yizhong technology co Ltd 地址： 广东顺德龙江龙峰大道38号 ----- 全国统一热线： +400 865 5603 ----- 邮箱： sales@dirchina.cn



Product Specification

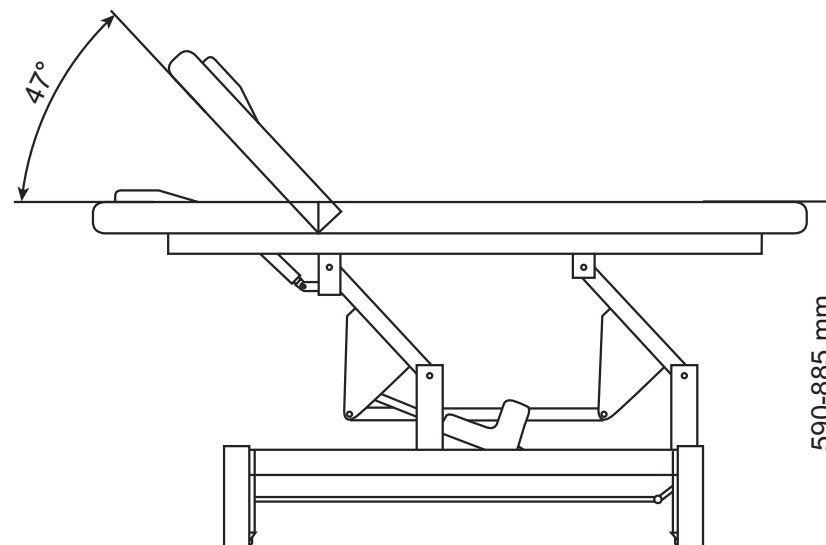


Measurement: L1850*W720*H590(mm)
Hydraulic Height: 590 - 885 mm
Net weight: 63 KG
Gross weight: 88 KG
Maximum load: 150 KG
Rated voltage: 220-240V, 50Hz
Output voltage: DC 24V

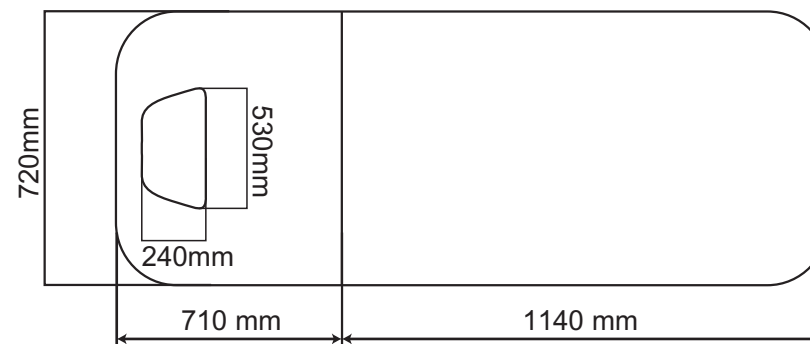


Product Specification

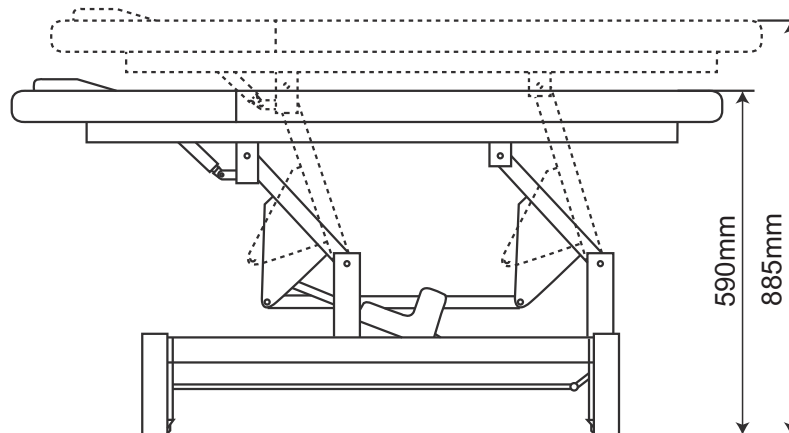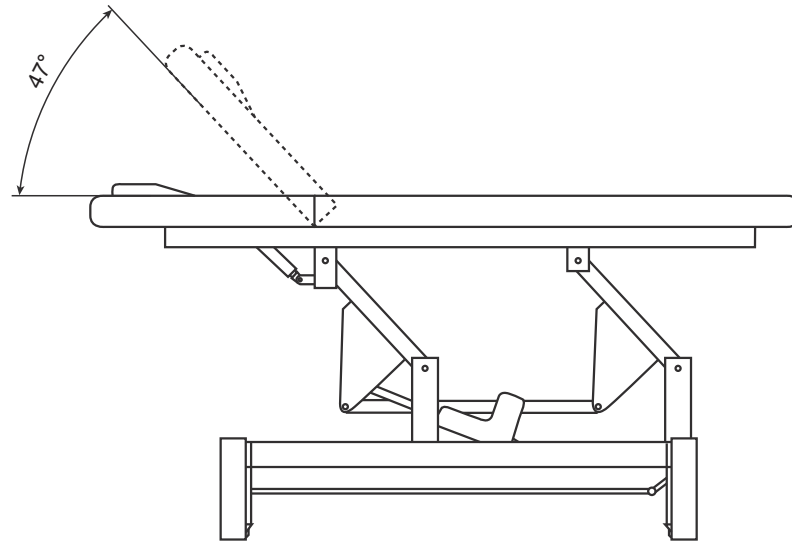

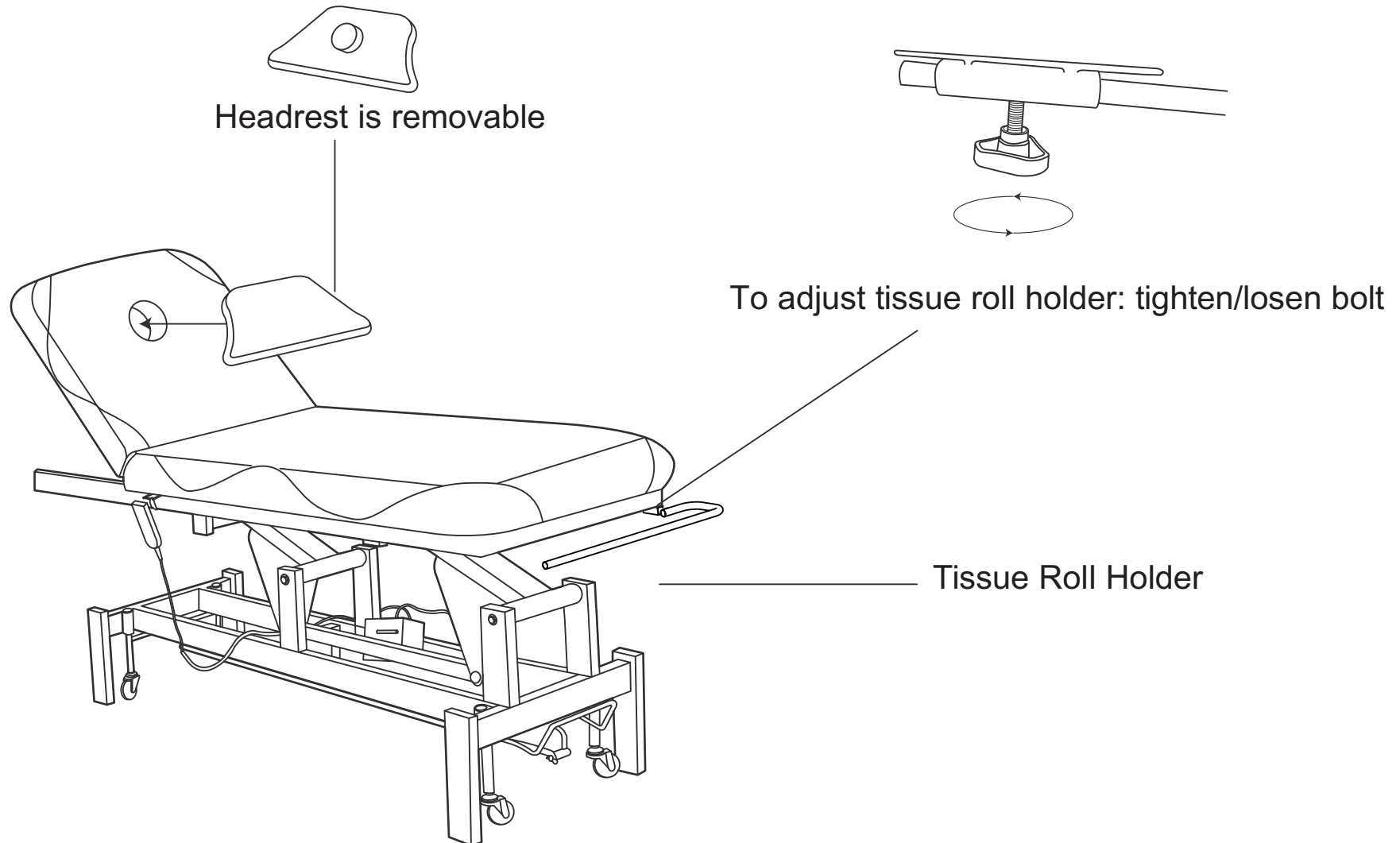


Product Specification

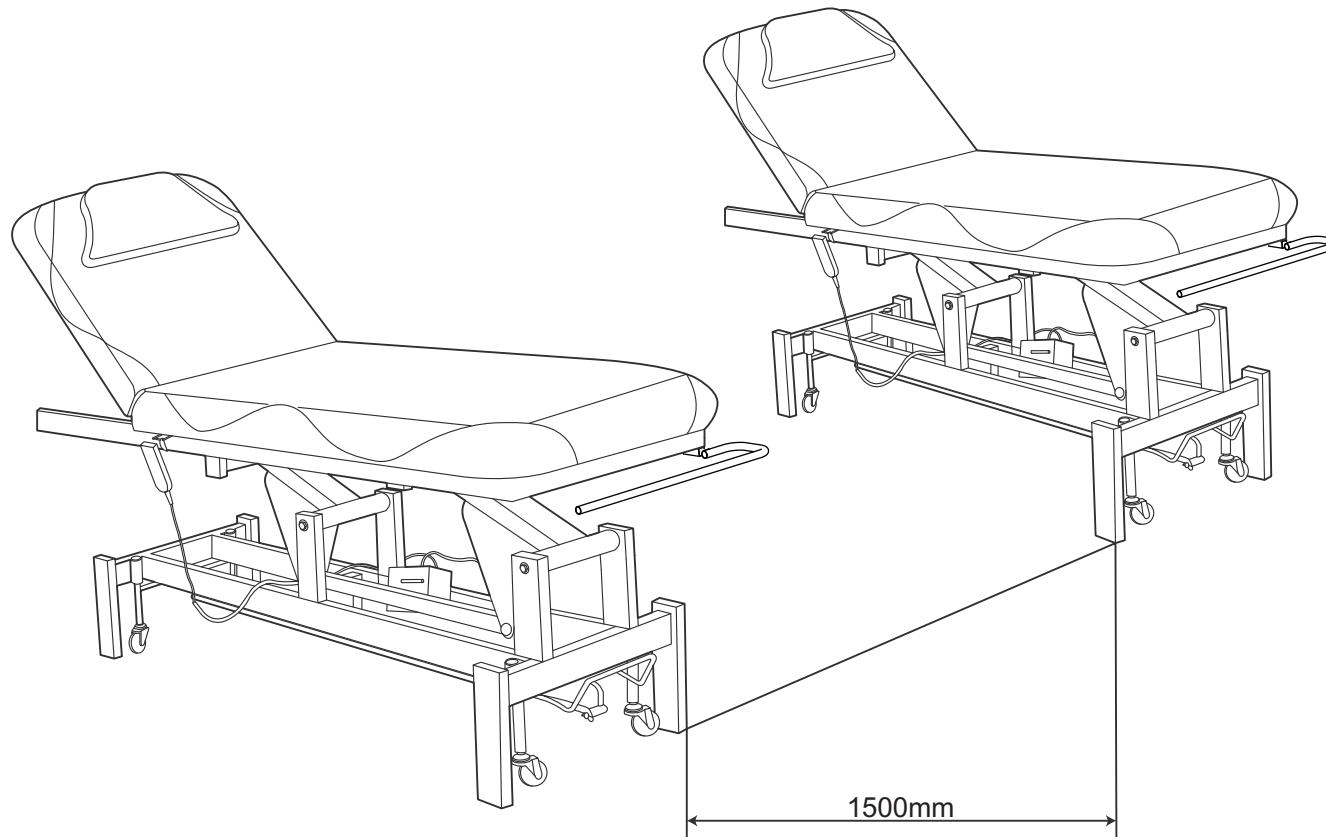


Recommended distance: 1.5 meter apart

Wheel Lever


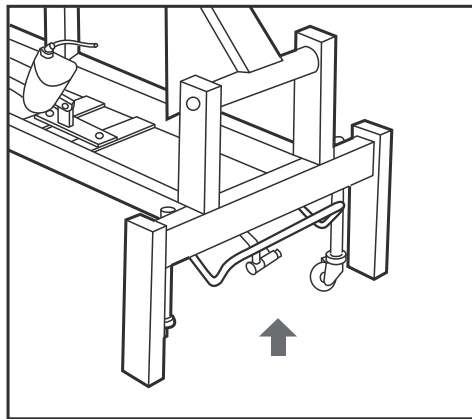


Push lever up to lock wheels

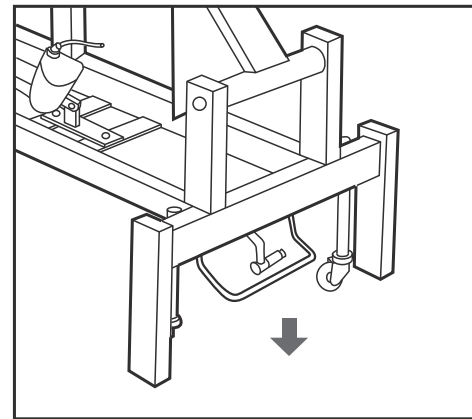


Put lever down to manover facial bed.


Operation Instructions

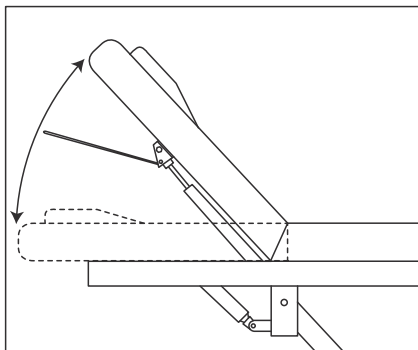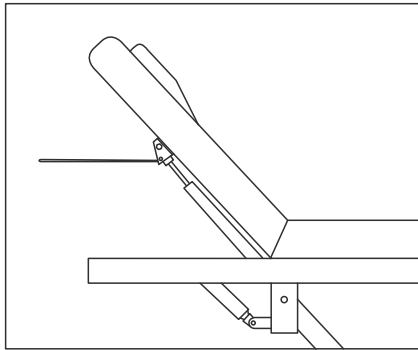

Pull the adjustment lever up, hold the backrest to the position you prefer and release the lever to lock the backrest.

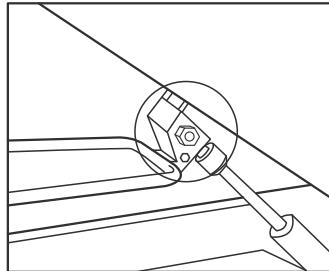


Please do not tighten the screw too much in case the lever can not work properly.

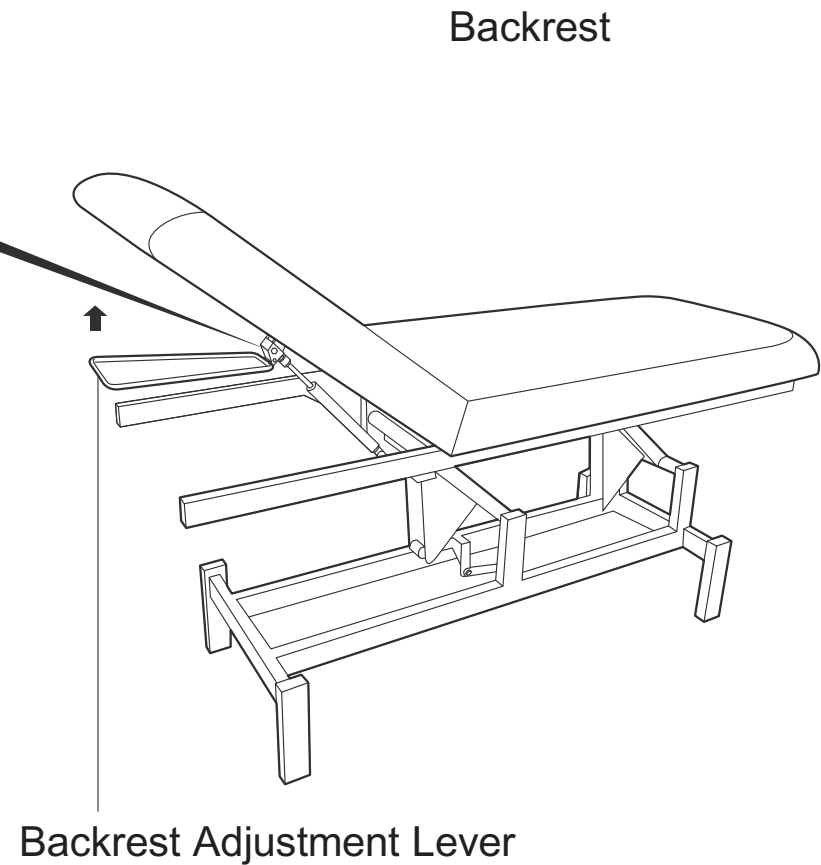

Control Panel

Control Panel (Hand)

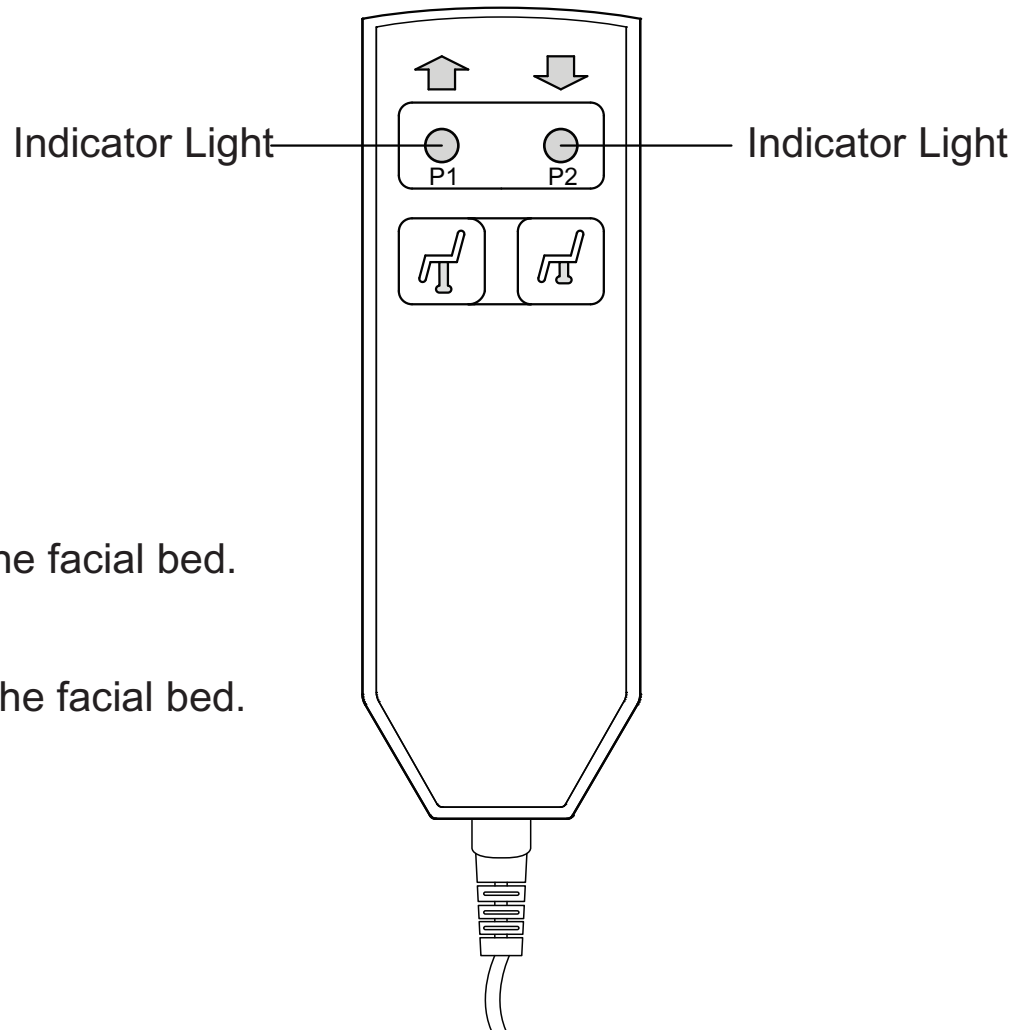


Press the button to raise the facial bed.



Press the button to lower the facial bed.


Control Panel (Foot)

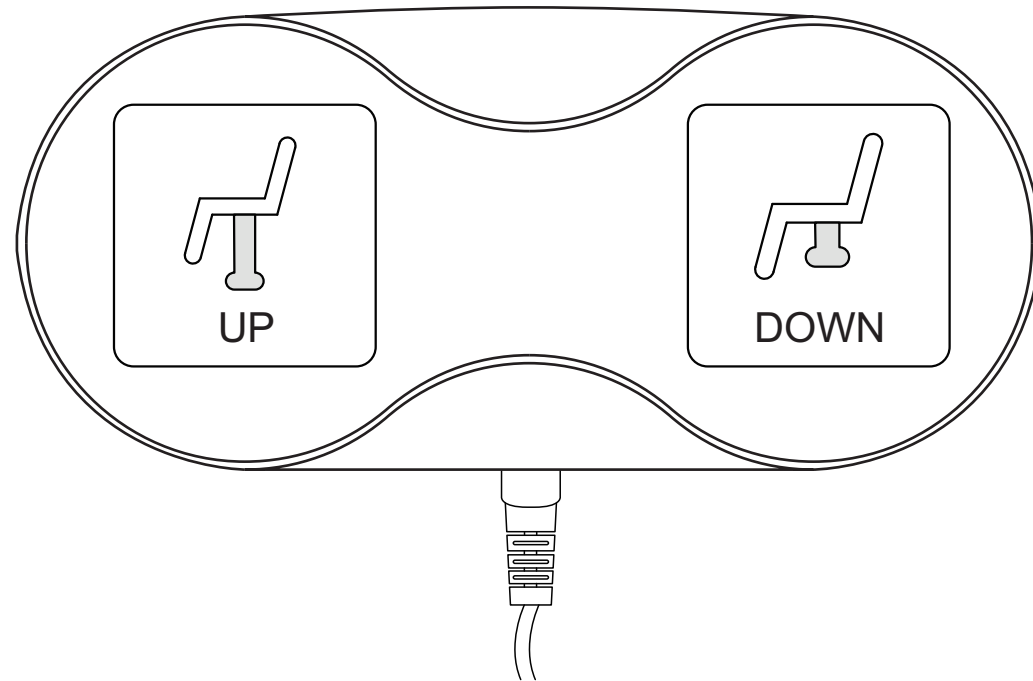


Push the button by foot to raise the facial bed.




Push the button by foot to lower the facial bed.

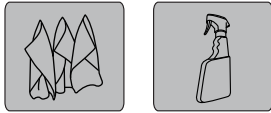


Wipe clean with a cloth dampened in a mild leather cleaner.
Wipe dry with a clean cloth.

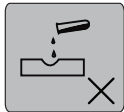

Avoid the use of abrasives, bleaches or solvents.

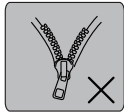

Never remove and wash faux leather covers.

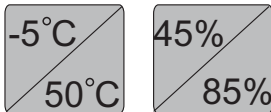

Storage and using environment

Please do not place this product under strong light or in moist environment, to avoid the damage on product. It can be used in the temperature of $-5^{\circ}\text{C}\sim 50^{\circ}\text{C}$ and humidity of $45\%\sim 85\%$




After an initial period of use, check all fastenings for any loosened screws etc. If found, please tightened appropriately.
Please keep children away from the product for safety reason.
If the power cord has any damage, please contact the technical support team.

If you meet problems while using this product, please try the given solutions.


Problems	Causes	Solutions
When press the remote control, The bed doesn't move.	1. Electrical plug is not inserted well. 2. Transformer links to each plug which is not inserted well.	1. Pull out plug and insert it again. 2. Pull out the transformer plug under the bed and insert it again.
When clients use the bed, the bed shakes or makes noise.	Screws come loose.	Tighten each screw.

If there is still a problem after trying the solution given in this troubleshooting guide, please stop using the product and contact our customer service team on +44(0)121 643 3877


AFTERCARE INSTRUCTIONS

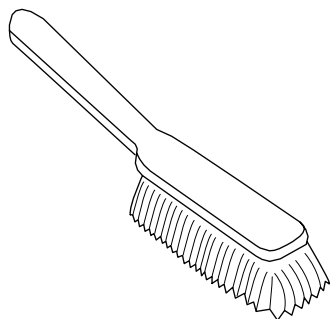


Brush away excess hair

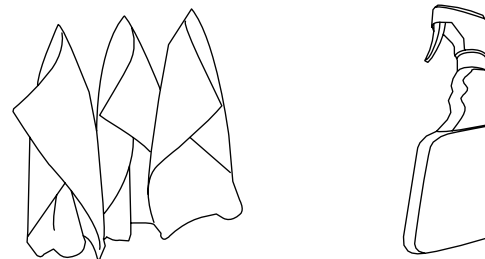

Wipe clean with a cloth dampened in a mild leather cleaner or wipe dry with a clean cloth

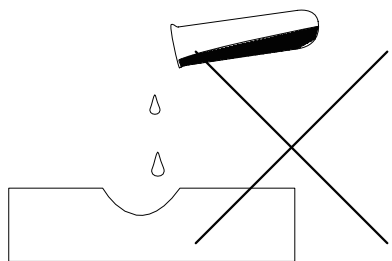

Avoid the use of abrasives, bleaches or solvents

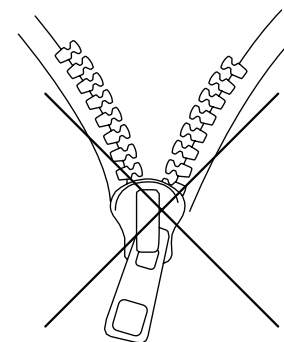

Never remove and wash leather covers

After an initial period of use, check all fastenings for any loosened screws etc. If found, please tightened appropriately

CONTACT US

DIR BIRMINGHAM (HEADQUARTERS)

66 Rea Street South
Birmingham
B5 6LB
UK
Sales: +44(0)800 085 3667
Customer Service: +44(0)121 6434777
Fax: +44(0)121 6333291
Email: sales@dirgroup.co.uk
Website: www.dirgroup.co.uk

DIR LONDON

Unit 1 River Road Business Park
33 River Road, Barking
Greater London IG11 0DA
UK
Sales: +44(0)20 8591 9310
Fax: +44(0)20 8594 4147
Email: sales.london@dirgroup.co.uk
Website: www.dirgroup.co.uk

DIR NEW YORK

24 Woodward Ave.
Suite C
Queens, NY 11385
USA
Sales: +1 (718)628-6668
+1 (800)728-2008
Fax: +1 855-628-5168
Email: sales@dirsalonfurniture.com
Website: www.dirsalonfurniture.com

DIR COLOGNE

Konrad-Adenauer-Straße 25
50996 Köln
Germany
Sales: + 49 (0) 221 / 4678 0417
Email: sales@dirgroup.de
Website: www.dirgroup.de

DIR IRELAND

Unit 5 Block E
Riverview Business Park
Nangor Road
Dublin 12
Republic of Ireland
Sales: +353 (01) 429 7969
Email: sales.ireland@dirgroup.ie
Website: www.dirgroup.ie

DIR CHINA

广东伊众健康科技有限公司
Guangdong Yizhong technology co Ltd
广东顺德龙江龙峰大道38号
全国统一热线：
+400 865 5603
邮箱: sales@dirchina.cn
官网：www.dirchina.cn

If you have any enquires regarding the product warranty, please contact our customer services team.

www.dirgroup.co.uk