


INK

8103

INSTRUCTION MANUAL

EN

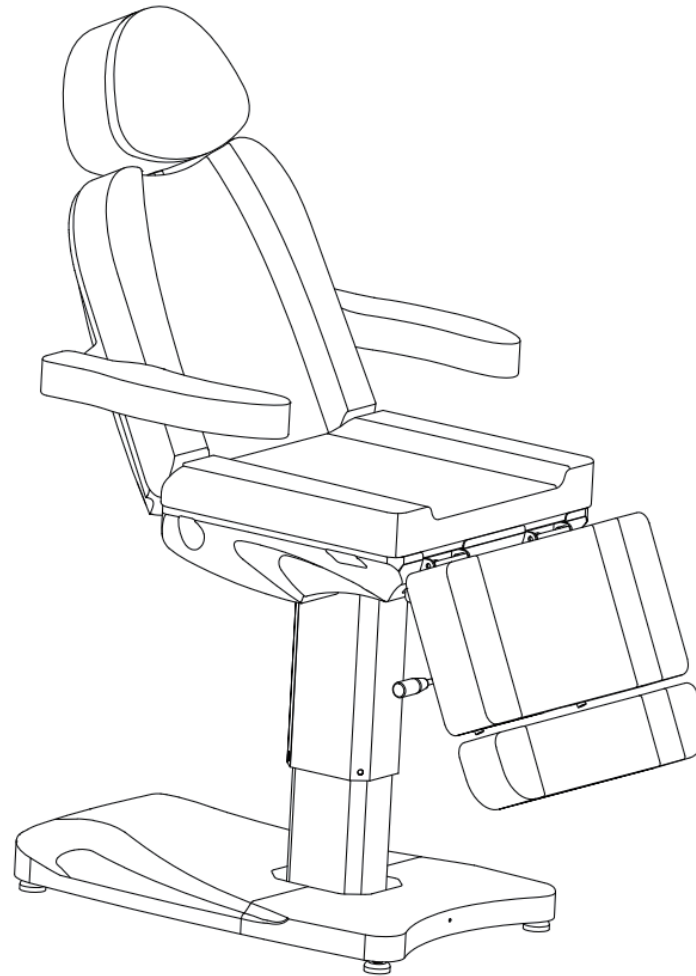



CONTENTS

Assembly Instructions	3
Technical data	3
List of accessories	4
The height adjustment	8
Attentions	10
Aftercare Instructions	12
Contact Us	13

Please read the operation manual before using the product.

Please put the operation manual in the place easy for you to read and you can read it at any time.

Product specifications

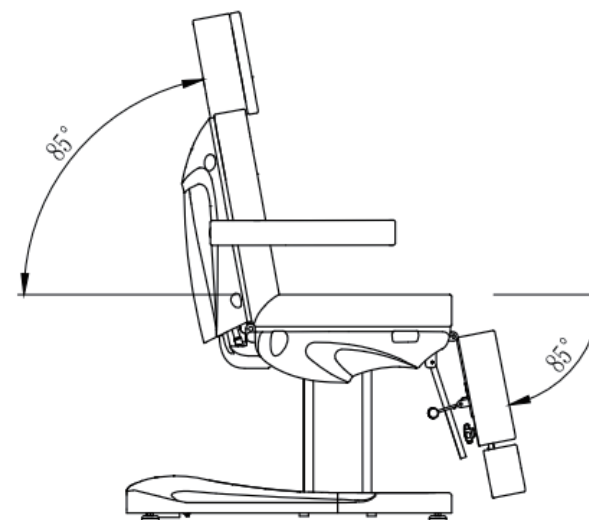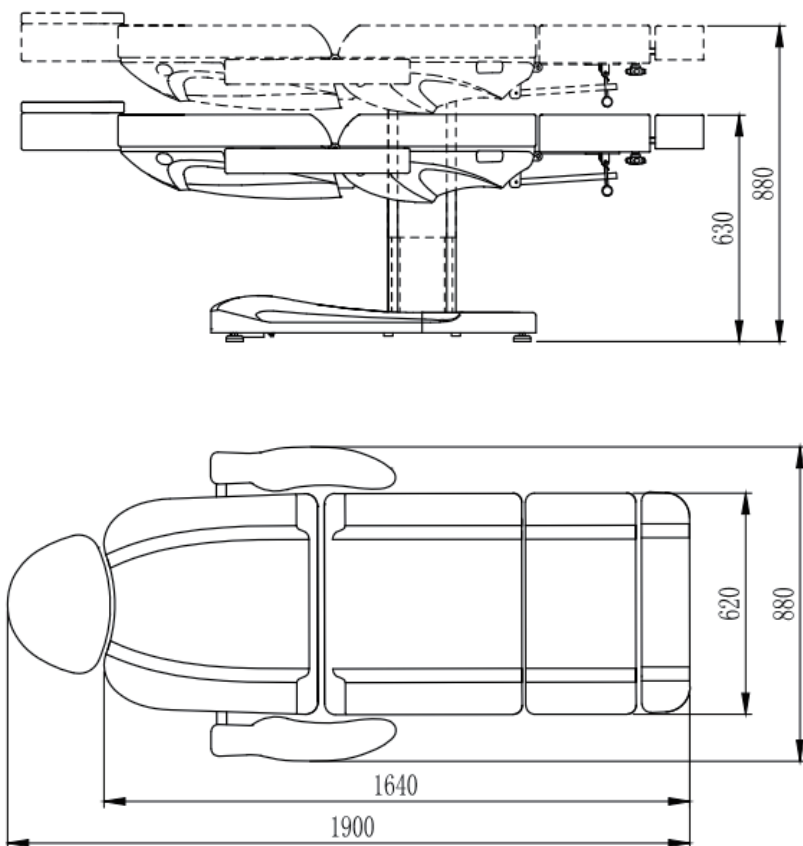






ASSEMBLY INSTRUCTIONS

Technical data


Measurement	1900*880*630mm
Maximum load	180kg
Net weight	66kg
Adjustable range of height	630 ~ 880mm
Input voltage	AC220~240V, 50Hz
Output voltage	DC 24V

List of accessories

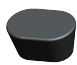
Accessories	Picture	QTY
Left Armrest		1
Right Armrest		1
Headrest		1
Extra Soft headrest		1
Headrest stuffed		1

Assembly instructions

- (1) Install the pillow, connect it to the bed, loose the button to adjust the height of the pillow, as shown in picture.
- (2) Install the armrests, connect the bar of left and right armrests with the inner pin of the bed, Rotate the armrests by 90 degrees counterclockwise, as shown in picture

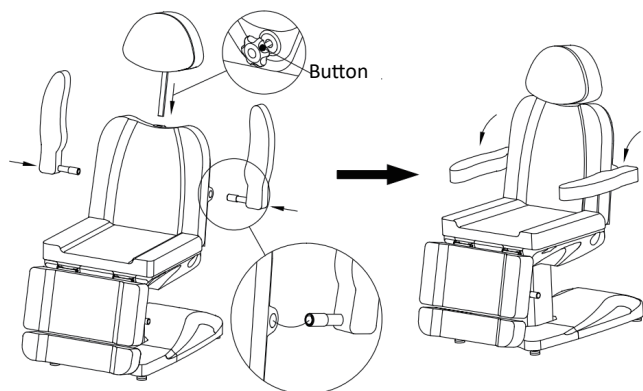


Operating instruction Remote Panel

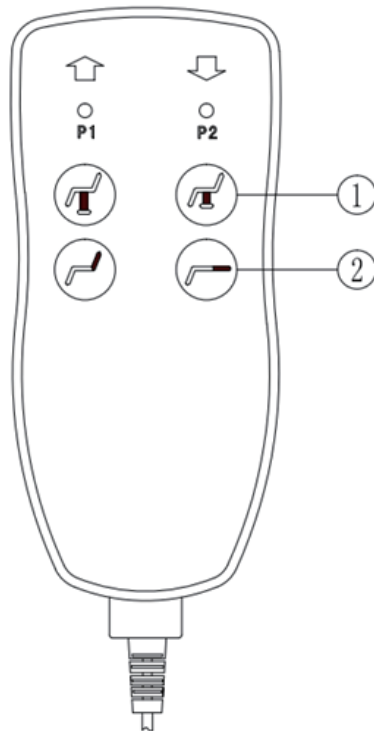

Introduction for remote panel

Press button ① to adjust the whole bed up and down.
Press button ② to adjust the backrest up and down.


Introduction for footrest panel

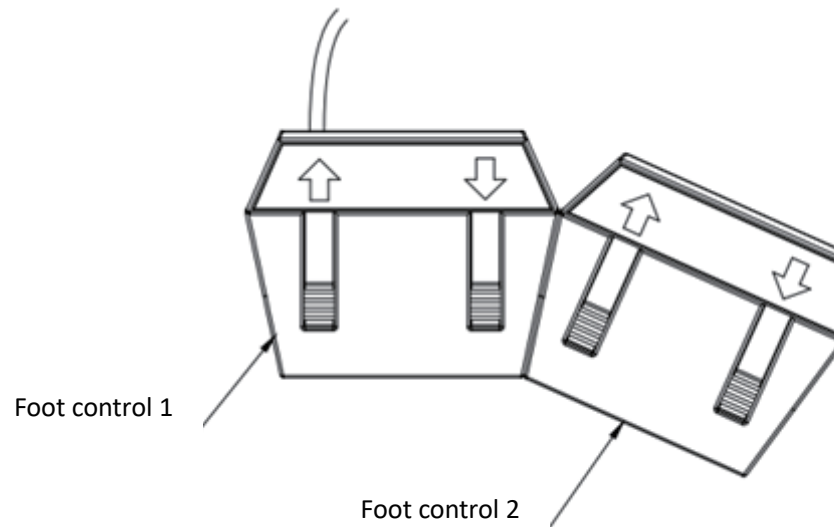


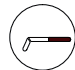


Press foot control 1 to adjust the whole bed up and down.
Press foot control 2 to adjust the backrest up and down.


The height adjustment

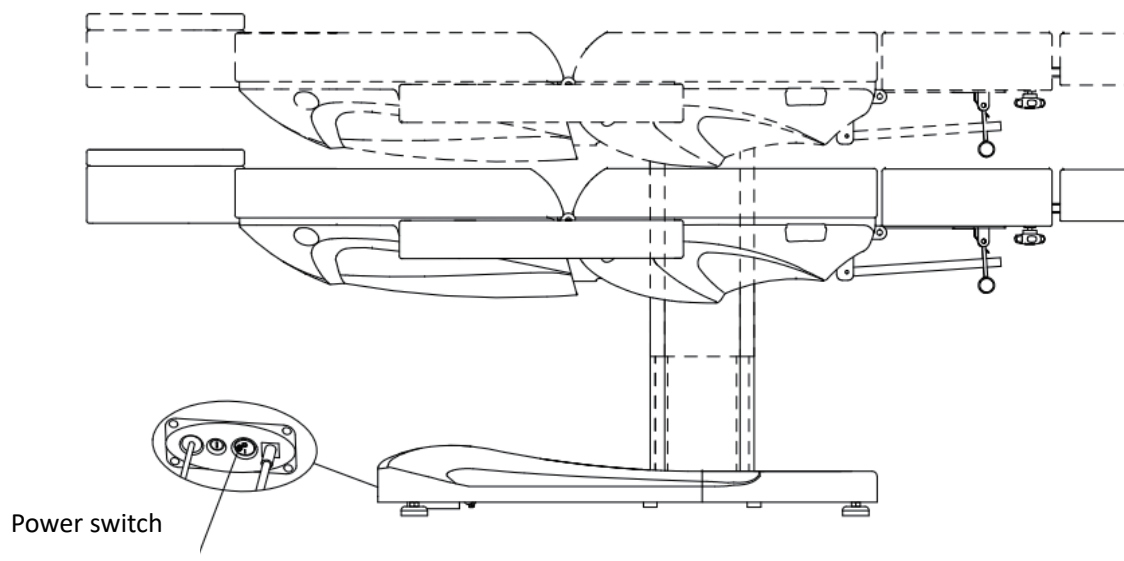- a. Turn on the power, then press remote button to rise up or press button to fall down the whole bed
- b. Step on the foot control 1 to rise up or fall down the whole bed by or button.




The backrest adjustment

- Press remote button  or  to adjust backrest up and down+
- Step on the foot control 2 to adjust backrest up and down by  or  button.

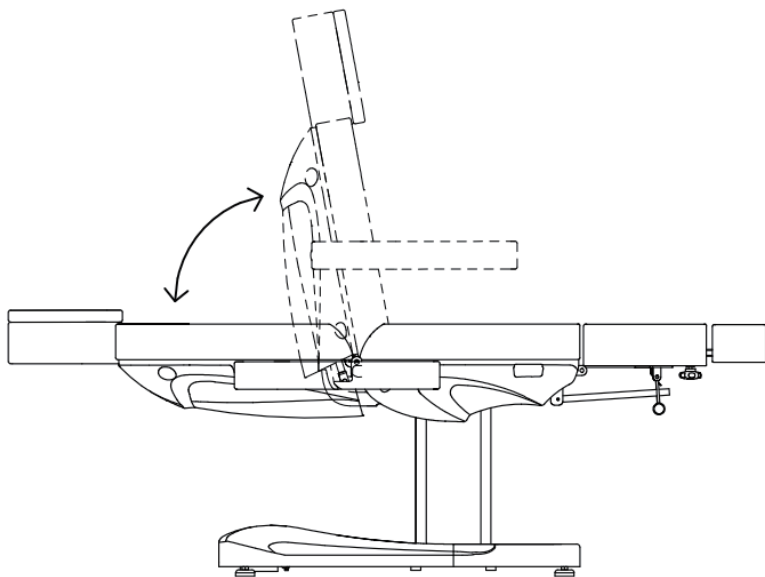

The footrest adjustment


Adjust footrest angle by pulling adjusted handle and press footrest

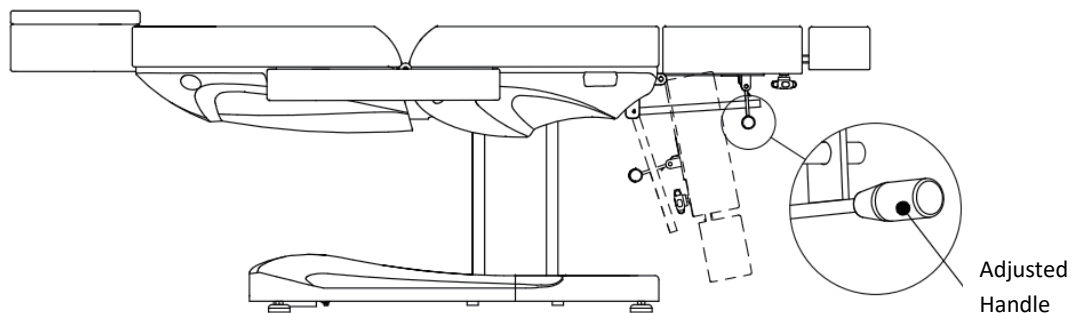

ASSEMBLY INSTRUCTIONS


Footrest extension adjustment

To adjust the extension of the footrest by rotating the knob under the footrest, as below picture

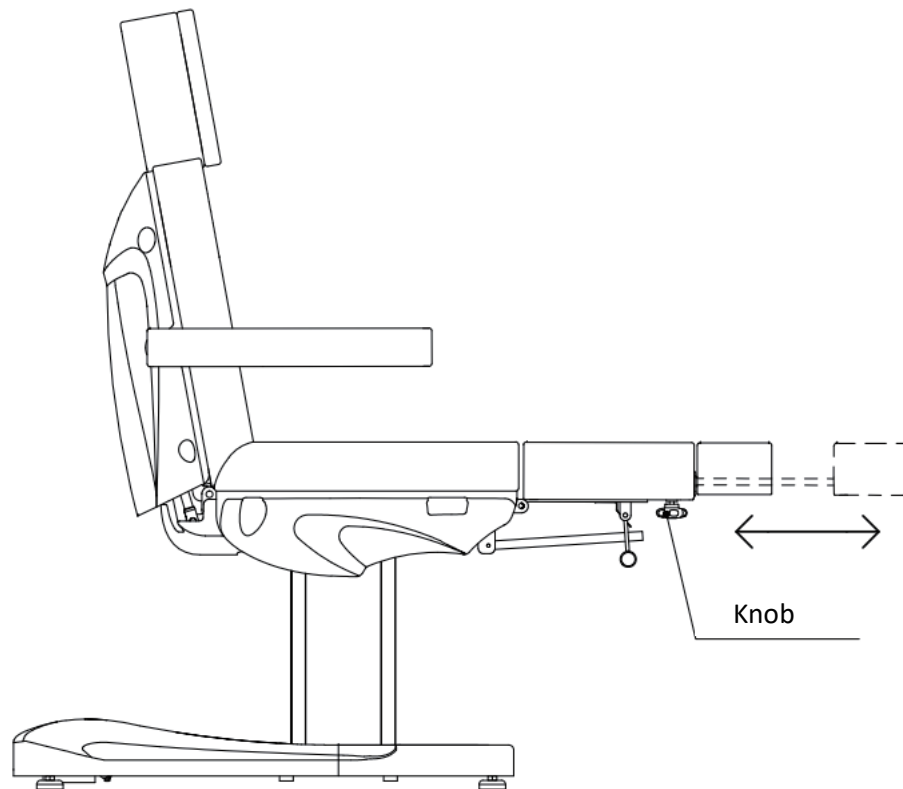

The suggestive instructions

1. This electric massage bed uses a three-wire power cable. Meanwhile it should be inserted into the three-hole power socket with earth wire. And it should be assured that it can be connected to the earth wire smoothly so that it can prevent people from getting electric shock while functioning normally.
2. All the plugs, buttons and other components connected to the power cable must not dip in the water or other liquid.
3. It is forbidden to put your hands, feet, head or other part of the body into the gap ①②③④ while massage chair is running when you press the manual control panel
4. It is prohibited to use when plugs, buttons and other components connected to the power cable are broken.
5. It should be noticed that the frequency is 6min/h when the motor of this massage chair is used.
6. The maintenances should not be dealt with any person without official training.

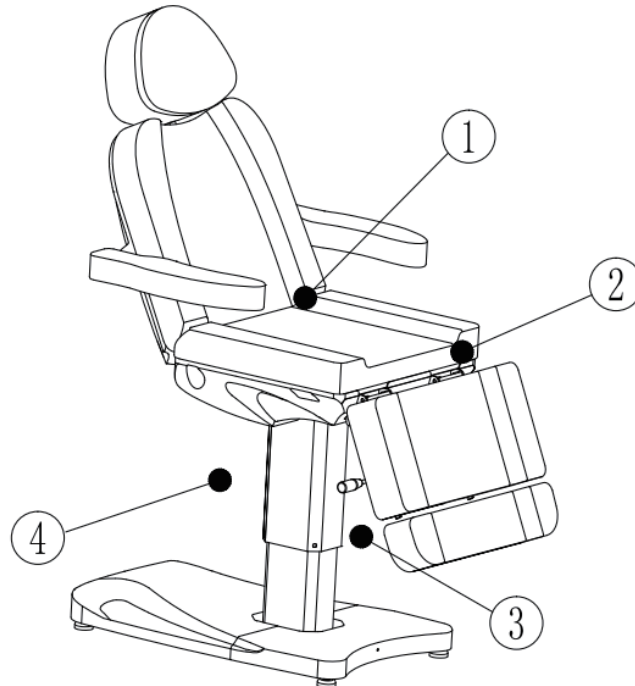

ASSEMBLY INSTRUCTIONS

Attentions

1. Someone with osteoporosis or heart problem should not use this massage chair.
2. Women being pregnant or in menstruation should not use this massage chair.
3. Someone injured or with disabilities should not use this massage chair.
4. Children under 14 years old without any supervision and someone unconscious should not use this massage chair.
5. Breakages of the power cable and plug should be eliminated before it's normally used. Breakages should be dealt with by someone professionals or service providers.
6. The voltage should accord with the safety voltage before it's officially used. And the wrong voltage may lead to the breakdown of the product.
7. Wet hand is not allowed to touch power to avoid shock.
8. The product should not plug in before it's already installed.
9. The power cable should be put away so that pedestrian will not trip up and fall.
10. The appearance of the product should be cleaned by the use of the soft towel with diluted neutral detergent.
11. It should be unplugged when it's not used.
12. Do not dismount this product without any authorization. Please inform the official service providers to deal with the maintenances

Maintenance

1. Daily cleaning
You can wipe out the dust by using soft dry towel.
2. Eliminating the stubborn dirt
As we all know, it is hard to eliminate the stubborn dirt by using normal dry towel. In this case, you can use soft towel with diluted neutral detergent to clean it first. And then you can use dry towel to scrub.
3. It's prohibited to use benzene, diluent, gasoline or other solvent.
These solvents may damage the leather or it may cause the corrosion of the products' appearance.
4. Storage and operating environment
Don't put the product under the strong sunlight and moisture environment. Otherwise it may cause the damages of the product.
5. It is allowed to use this product under the temperature from -5°C to 50°C and the humidity from 45% to 85%.


Fault diagnosis

If there is something wrong when you use the product, please try to check and eliminate the wrong operation as followings.


Phenomenon	Diagnosis	Operation of elimination
Press the pedal but without any responses.	<ol style="list-style-type: none"> 1. Controller power switch start or not. 2. To check if the pedal has reached to maximum. 3. To check if the power plug is connected. 4.To check if each connecting plug of the transformer is connected. 	<ol style="list-style-type: none"> 1.Try to re-start the controller power switch. 2. Try other buttons workable or not. 3.Disconnect the power plug and re-plug it again. 4. Re-plug and connect it again.
Letting out noises and shaking while going up and down	Screw loosen	Tighten all the screws
The light is still off although the switch is on.	The fuse has burned out.	Replace the fuse.

Remarks: It will be not informed if the improvements of the product do not influence appearance and function.


AFTERCARE INSTRUCTIONS

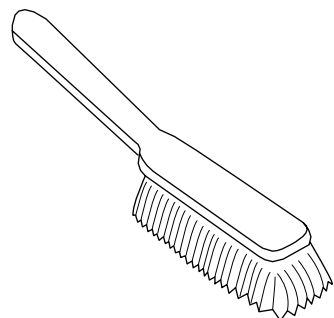


Brush away excess hair

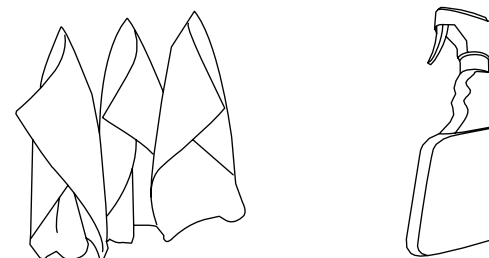

Wipe clean with a cloth dampened in a mild leather cleaner or wipe dry with a clean cloth

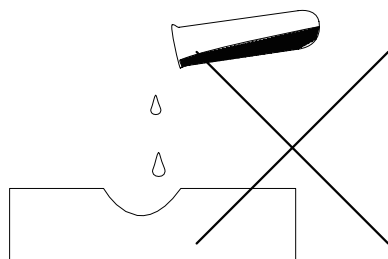

Avoid the use of abrasives, bleaches or solvents

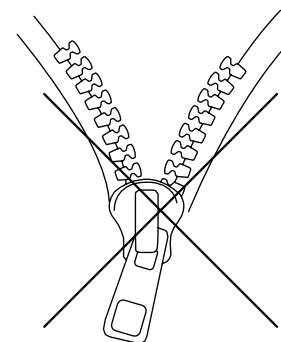

Never remove and wash leather covers

After an initial period of use, check all fastenings for any loosened screws etc. If found, please tightened appropriately

DIR BIRMINGHAM (HEADQUARTERS)

66 Rea Street South
Birmingham
B5 6LR
UK
Sales: +44(0)800 085 3667
Customer Service: +44(0)121 6433877
Fax: +44(0)121 6333291
Email: sales@dirgroup.co.uk
Website: www.dirsalonfurniture.uk

DIR LONDON

Unit 1 River Road Business Park
33 River Road, Barking
Greater London IG11 0DA
UK
Sales: +44(0)20 8591 9310
Fax: +44(0)20 8594 4147
Email: sales.london@dirgroup.co.uk
Website: www.dirsalonfurniture.uk

DIR NEW YORK

24 Woodward Ave.
Suite C
Queens, NY 11385
USA
Sales: +1 (718)628-6668
+1 (800)728-2008
Fax: +1 855-628-5168
Email: sales@dirsalonfurniture.com
Website: www.dirsalonfurniture.com

DIR COLOGNE

Konrad-Adenauer-Straße 25
50996 Köln
Germany
Sales: + 49 (0) 221 / 88888.128
Fax: + 49 (0) 221 / 88888.122
Email: sales@dirgroup.de
Website: www.dirgroup.de

DIR IRELAND

Unit 5 Block E
Riverview Business Park
Nangor Road
Dublin 12
Republic of Ireland
Sales: +353 (01) 429 7969
Email: sales.ireland@dirgroup.ie
Website: www.dirgroup.ie

DIR CHINA

广东伊众健康科技有限公司
Guangdong Yizhong technology co Ltd
广东顺德龙江龙峰大道38号
全国统一热线 :
+400 865 5603
邮箱: sales@dirchina.cn
官网 : www.dirchina.cn

If you have any enquires regarding the product warranty, please contact our customer services team.

www.dirsalonfurniture.uk