


Pavo

8709

INSTRUCTION MANUAL

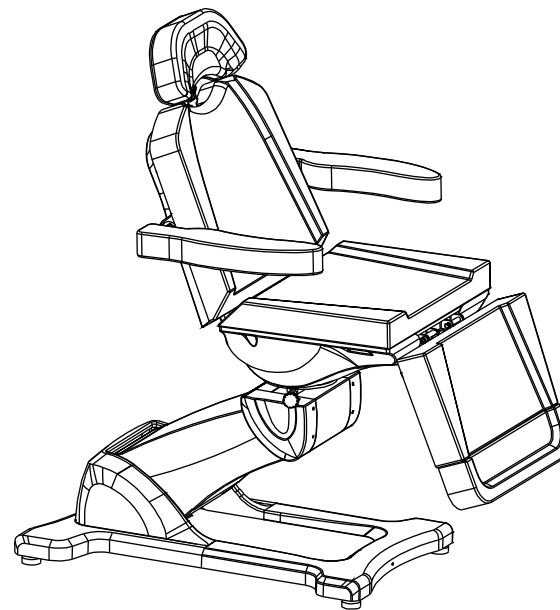

EN

CONTENTS

Product Introduction	3
Warranty	4
Product Information	5
Operation Instructions	6
Maintenance	13
Fault Diagnosis	14
Aftercare Instructions	15
Contact Us	16

Please read the operation manual before using the product.

Please put the operation manual in the place easy for you to read and you can read it at any time.

Product Name: Pavo
Product Code: 8709

Thank you for placing your order with the DIR . Our materials inc. faux leather etc, are specially sourced to provide quality and performance, as well as meeting UK/EU safety regulations to ensure customer satisfaction.

All of our facial beauty bed are produced with strong resilient faux leather, tested by our manufacturer to ensure strength, memory, heat resistance and stain proofing. We also fit our facial beauty bed with fire-retardant, pre-moulded memory foam to ensure continual use and sustained durability which we offer a one year warranty to ensure customer confidence in our products and our company.

WARRANTY

The warranty period is one year. DIR warrants to you, the original buyer, that goods shall be free of defects in material and/or workmanship attributable to manufacture for one year from date of buyer's receipt of product. Damage resulting from improper handling after merchandise is received, or from exposure to weather or extreme atmospheric conditions is not covered by the warranty.

For any other information please contact our support team:

DIR BIRMINGHAM	DIR LONDON	DIR NEW YORK	DIR COLOGNE	DIR IRELAND	DIR CHINA
66 Rea Street South Birmingham B5 6LB UK ----- Tel: +44(0)121 6434777 ----- Email: sales@dirgroup.co.uk	Unit 1 River Road Business Park 33 River Road, Barking Greater London IG11 0DA UK ----- Tel: +44(0)20 8591 9310 ----- Email: sales.london@dirgroup.co.uk	24 Woodward Ave. Suite C Queens, NY 11385 USA ----- Tel: +1 (718)628-6668 +1 (800)728-2008 ----- Email: sales@dirsalonfurniture.com	Konrad-Adenauer-Straße 25 50996 Köln Germany ----- Tel: + 49 (0) 221 4678 0417 ----- Email: sales@dirgroup.de	Unit 5 Block E, Riverview Business Park, Nangor Road, Dublin 12 Republic of Ireland ----- Tel: +353 (01) 429 7969 ----- Email: sales.ireland@dirgroup.ie	广东伊众健康科技有限公司 Guangdong Yizhong technology co Ltd 地址 : 广东顺德龙江龙峰大道38号 ----- 全国统一热线 : +400 865 5603 ----- 邮箱: sales@dirchina.cn

DIR

www.dirsalonfurniture.uk

Dimension: 1780X880X600mm

Minimum Height: 600mm

Maximum Height: 1000mm

N.W: 86kg



Max load capacity: 180kg (normal use)

Input voltage: AC 230V/50HZ


Output Voltage: DC 24V

ASSEMBLY INSTRUCTIONS

Product Specification

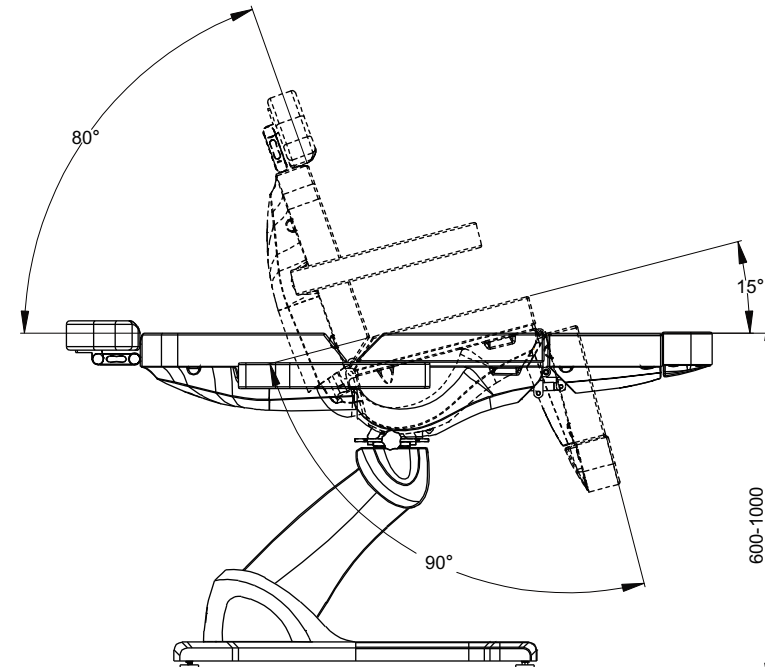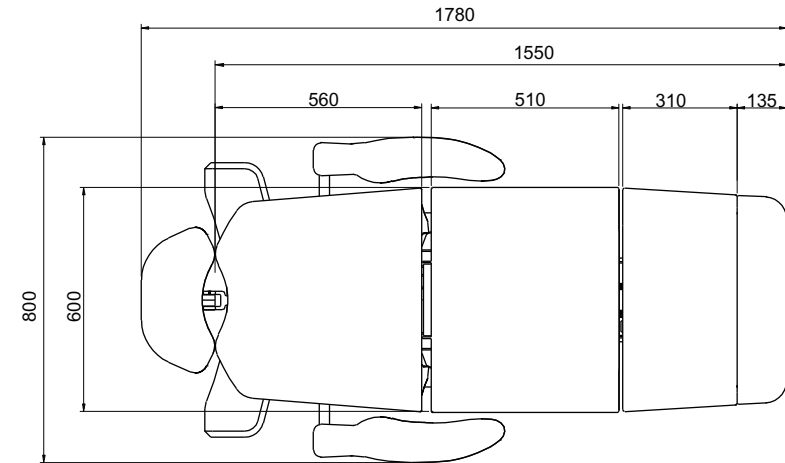



ASSEMBLY INSTRUCTIONS

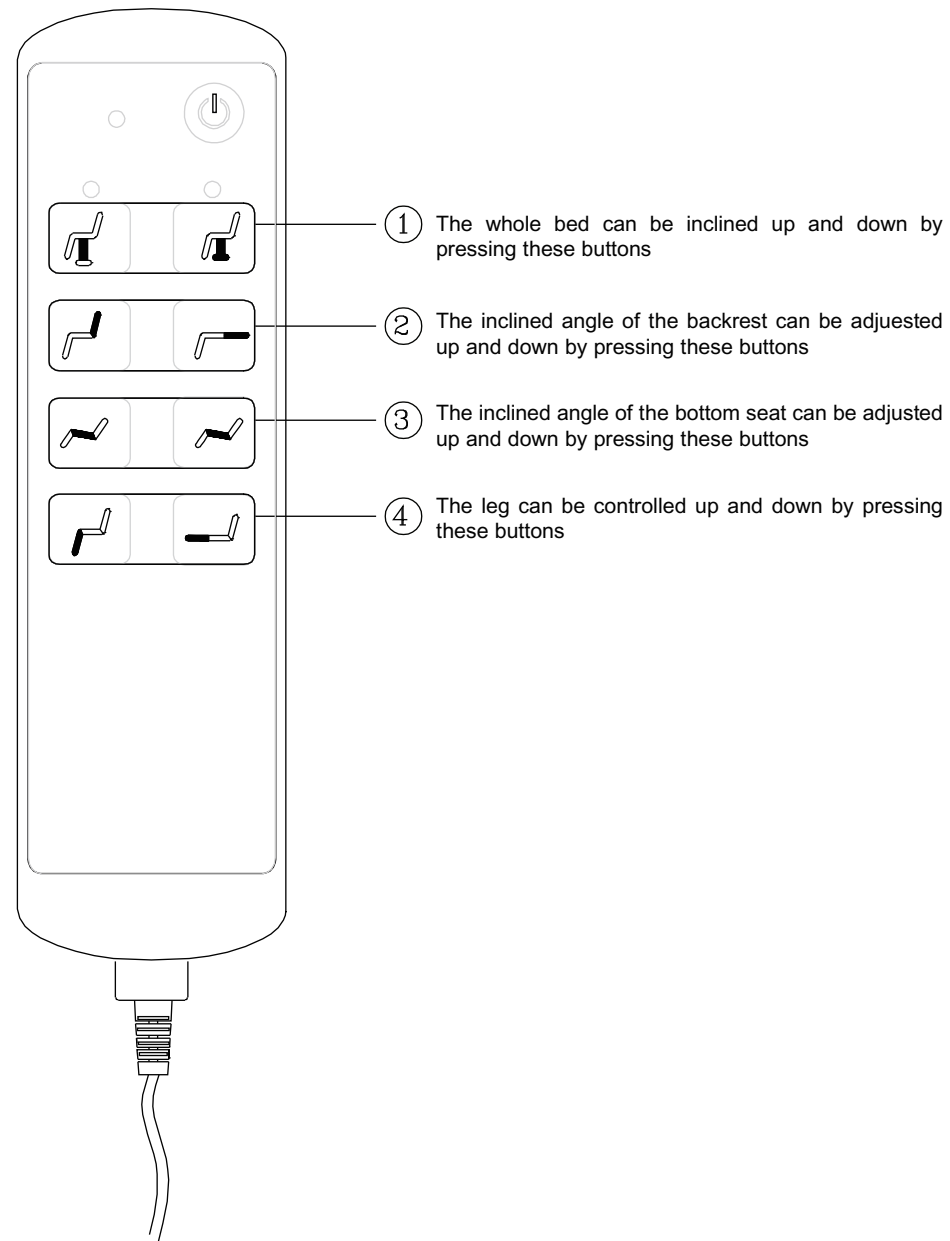

The instruction

(1) The height adjustment



The whole bed can be inclined up and down by pressing these buttons:

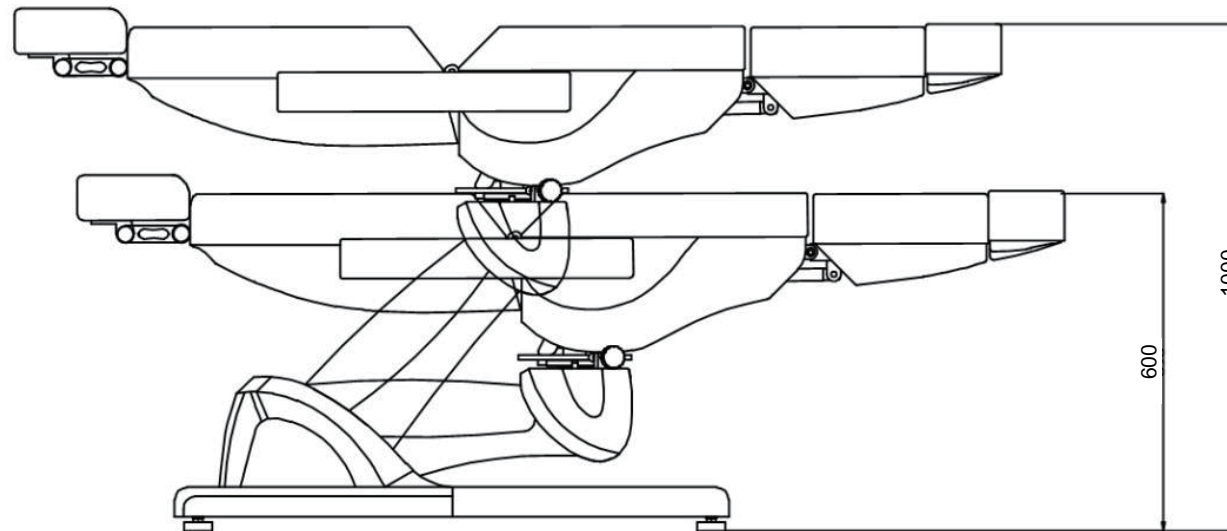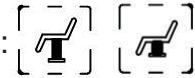

ASSEMBLY INSTRUCTIONS



(2) The backrest adjustment

The inclined angle of the backrest can be adjusted up and down by pressing these buttons:

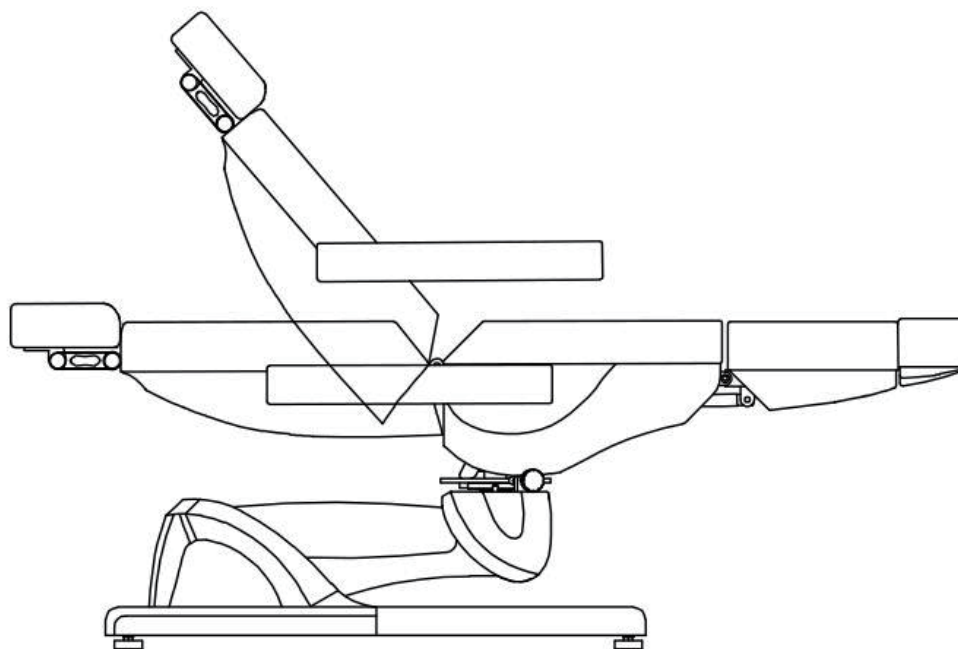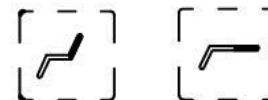

(3) The bottom seat adjustment


The inclined angle of the bottom seat can be adjusted up and down by pressing these buttons:

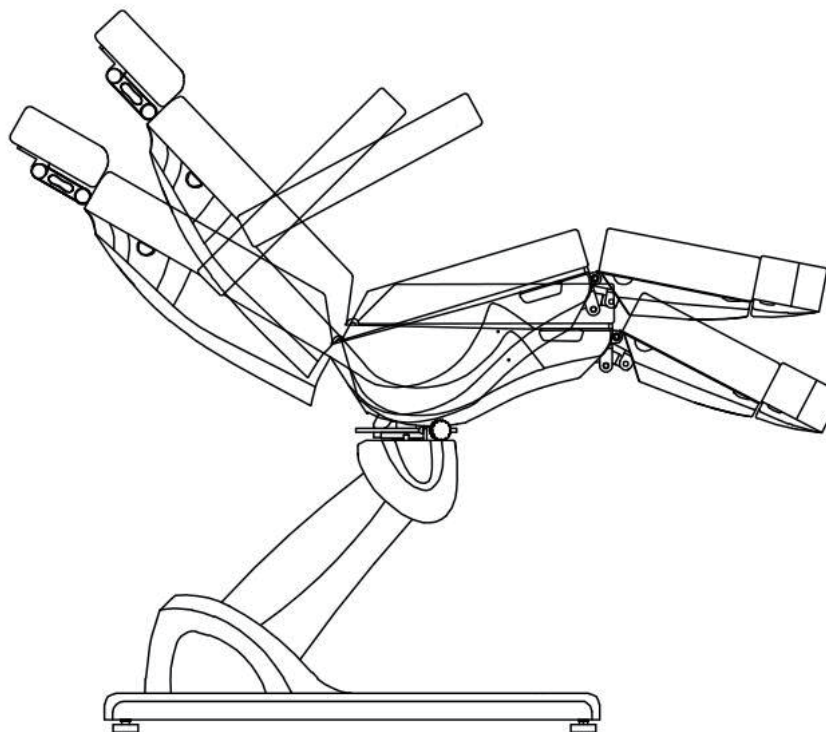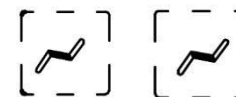

ASSEMBLY INSTRUCTIONS

(4) The footrest adjustment



The footrest can be controlled up and down by pressing these buttons:

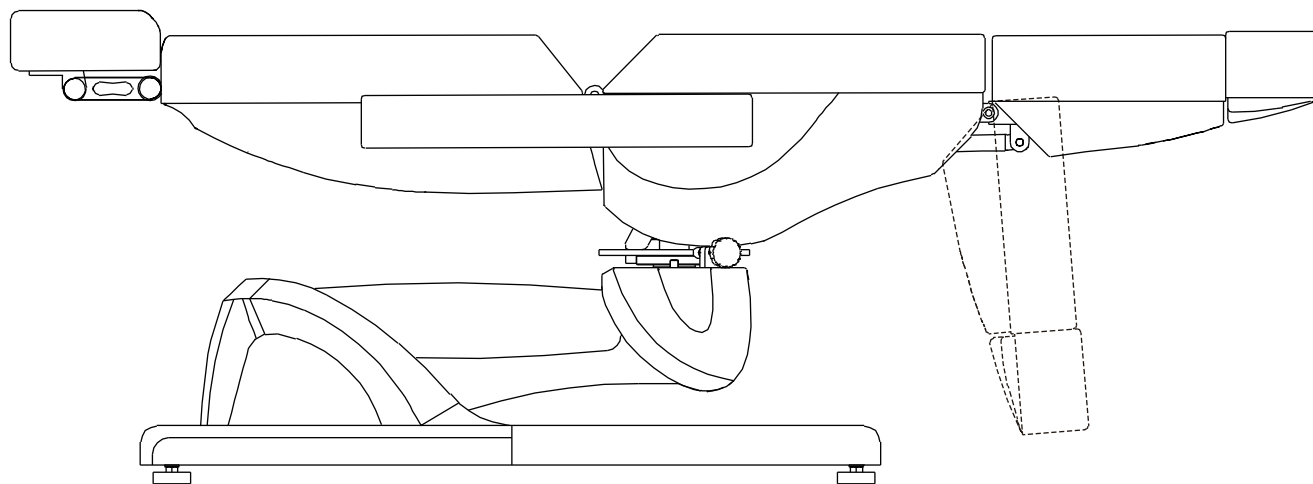

ASSEMBLY INSTRUCTIONS

(5) The pillow adjustment



The inclined angle of the whole pillow can be adjusted if you press the button A. And the retroflexion of the pillow can be adjusted individually if you press the button B.

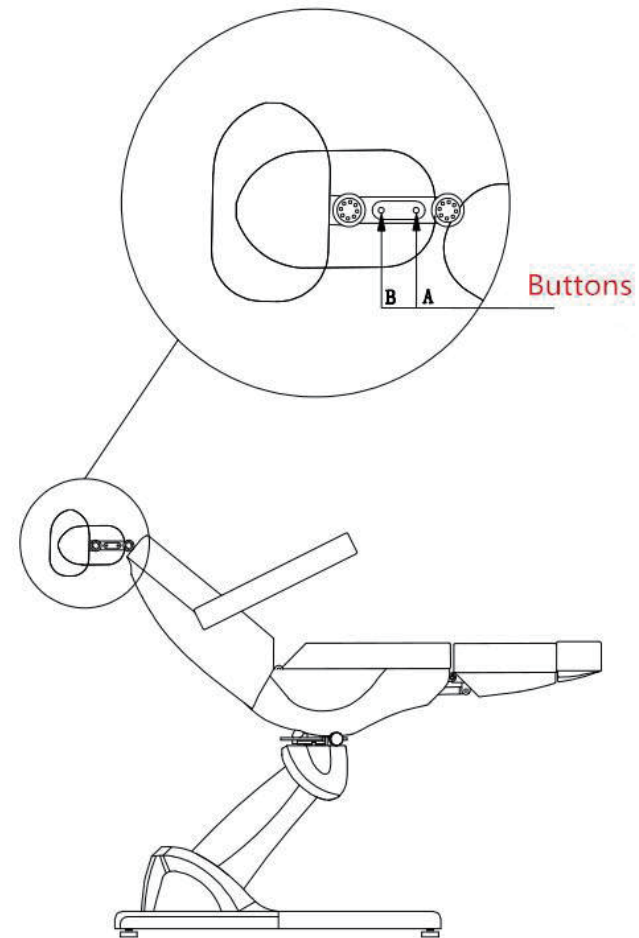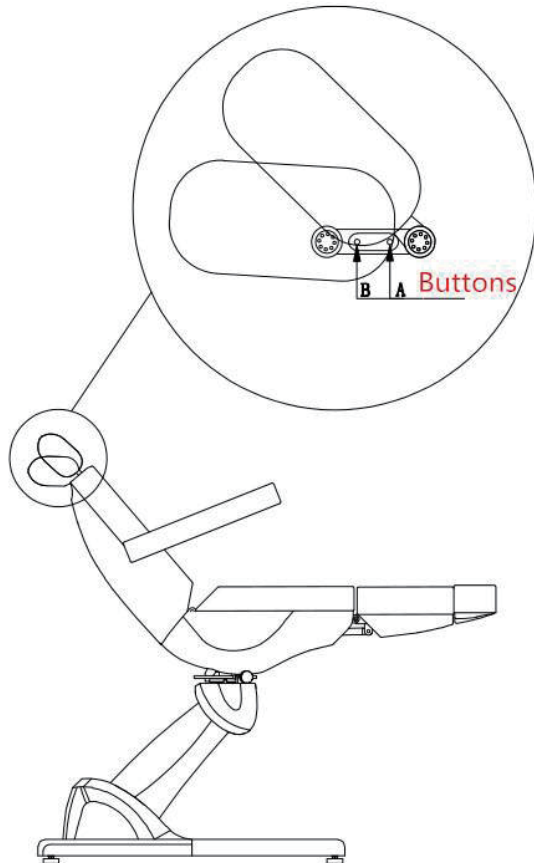

ASSEMBLY INSTRUCTIONS

(6) The rotation function

This beauty bed has a rotation function. First, you arrow shows you in the picture, and then you should move the surface of the beauty bed. When the beauty bed is rotated to $\pm 120^\circ$, no matter it is left side or right side, these two angles can be fixed because of the designed card slots at the inner turntable.

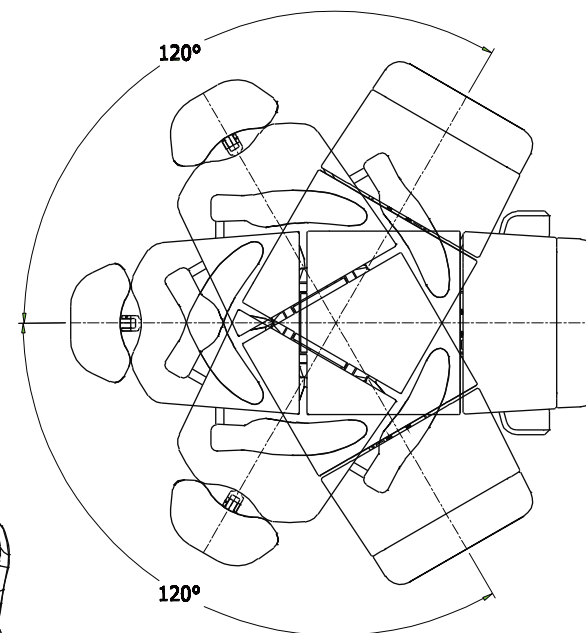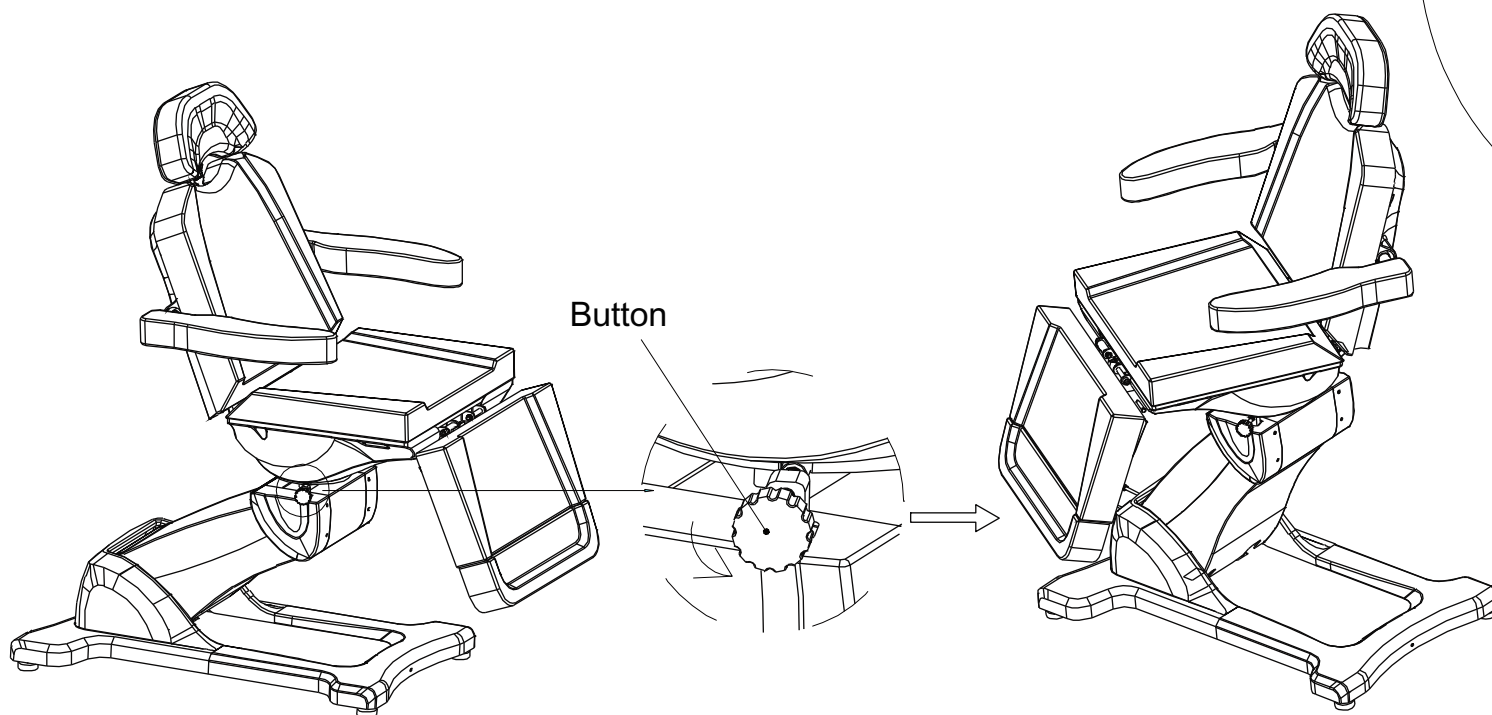

Maintenance

1. Daily cleaning

You can wipe out the dust by using soft dry towel.

2. Eliminating the stubborn dirt

As we all know, it is hard to eliminate the stubborn dirt by using normal dry towel. In this case, you can use soft towel with diluted neutral detergent to clean it first. And then you can use dry towel to scrub.

3. It's prohibited to use benzene, diluent, gasoline or other solvent.

These solvents may damage the leather or it may cause the corrosion of the products appearance.

4. Storage and operating environment

Don't put the product under the strong sunlight and moisture environment. Otherwise it may cause the damages of the product.

5. It is allowed to use this product under the temperature from 5°C to 50°C and the humidity from 45% to 85%.

ASSEMBLY INSTRUCTIONS


Fault diagnosis

If there is something wrong when you use the product, please try to check and eliminate the wrong operation as followings.


Phenomenon	Diagnosis	Operation of elimination
Press the button on the remote but without any responses.	1. The function of control button has reached to maximum. 2. To check if the power plug is connected. 3. To check if each connecting plug of the transformer is connected.	1. Try to press other control button. 2. Disconnect the power plug and connect it again. 3. Disclose the chassis. Discharge the plug and connect it again.
Letting out noises and shaking while going up and down	Screw loosen	Tighten all the screws

Remarks: It will be not informed if the improvements of the product do not influence appearance and function.


AFTERCARE INSTRUCTIONS

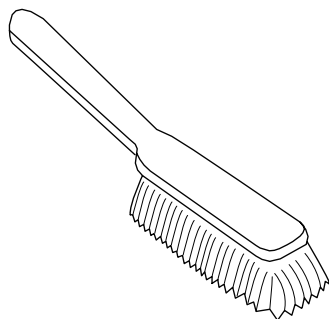


Brush away excess hair

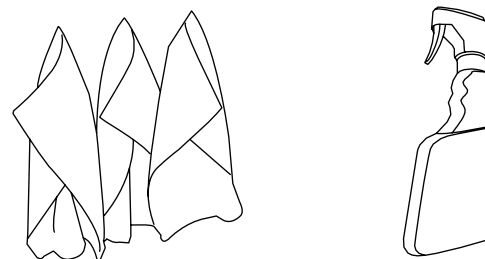

Wipe clean with a cloth dampened in a mild leather cleaner or wipe dry with a clean cloth

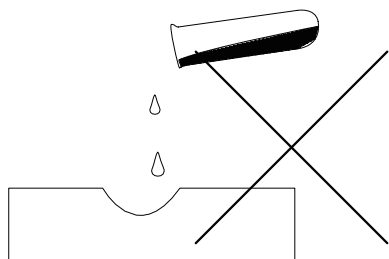

Avoid the use of abrasives, bleaches or solvents

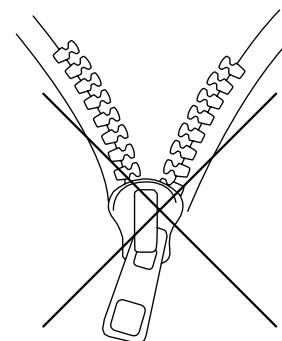

Never remove and wash leather covers

After an initial period of use, check all fastenings for any loosened screws etc. If found, please tightened appropriately

CONTACT US

DIR BIRMINGHAM (HEADQUARTERS)

66 Rea Street South
Birmingham
B5 6LB
UK
Sales: +44(0)800 085 3667
Customer Service: +44(0)121 6433877
Fax: +44(0)121 6333291
Email: sales@dirgroup.co.uk
Website: www.dirsalonfurniture.uk

DIR LONDON

Unit 1 River Road Business Park
33 River Road, Barking
Greater London IG11 0DA
UK
Sales: +44(0)20 8591 9310
Fax: +44(0)20 8594 4147
Email: sales.london@dirgroup.co.uk
Website: www.dirsalonfurniture.uk

DIR NEW YORK

24 Woodward Ave.
Suite C
Queens, NY 11385
USA
Sales: +1 (718)628-6668
+1 (800)728-2008
Fax: +1 855-628-5168
Email: sales@dirsalonfurniture.com
Website: www.dirsalonfurniture.com

DIR COLOGNE

Konrad-Adenauer-Straße 25
50996 Köln
Germany
Sales: + 49 (0) 221 / 88888.128
Fax: + 49 (0) 221 / 88888.122
Email: sales@dirgroup.de
Website: www.dirgroup.de

DIR IRELAND

Unit 5 Block E
Riverview Business Park
Nangor Road
Dublin 12
Republic of Ireland
Sales: +353 (01) 429 7969
Email: sales.ireland@dirgroup.ie
Website: www.dirgroup.ie

DIR CHINA

广东伊众健康科技有限公司
Guangdong Yizhong technology co Ltd
广东顺德龙江龙峰大道38号
全国统一热线 :
+400 865 5603
邮箱: sales@dirchina.cn
官网 : www.dirchina.cn

If you have any enquires regarding the product warranty, please contact our customer services team.

www.dirsalonfurniture.uk